



**CE**  
certification

# COVID-19 Antigen Rapid Test Cassette (Nasal Swab)

- Designed for qualitative detection of SARS-CoV-2 antigen in samples

## ■ Antigen rapid tests


Ag rapid tests are useful for detecting SARS-CoV-2 antigens, most often nucleocapsid, produced by the replicating virus in respiratory secretions.

Based on the above principle, COVID-19 Antigen Rapid Test Cassette (Nasal Swab) manufactured by Clongene is intended for the qualitative detection SARS-CoV-2 nucleocapsid antigens from individuals who are suspected of COVID-19 by their healthcare provider.

## ■ Kit Contents


Work Station	Extraction Reagent	Test Cassette	Extraction Tube & Dropper Tip	Sterilized Swab

## ■ Principle

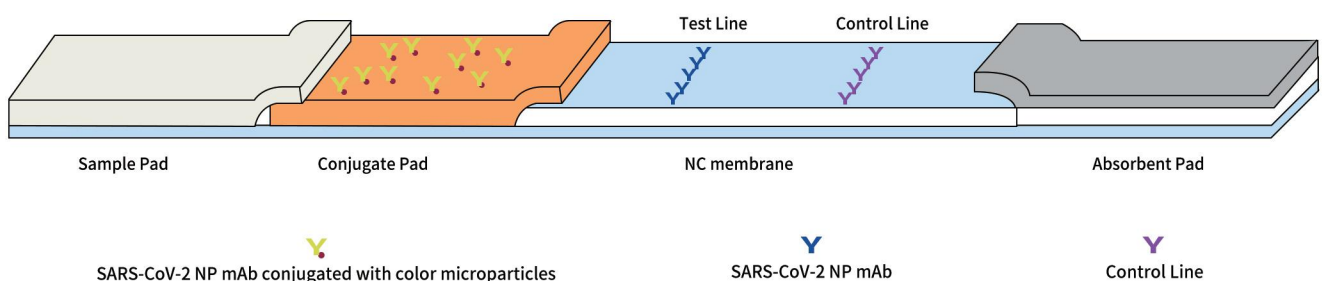


## ■ Procedure

### sample collection

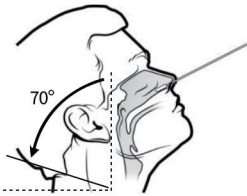


Insert swab about 2.5 cm (1 inch) into nostril until resistance is met at turbinates and rotate the swab.

### test procedure

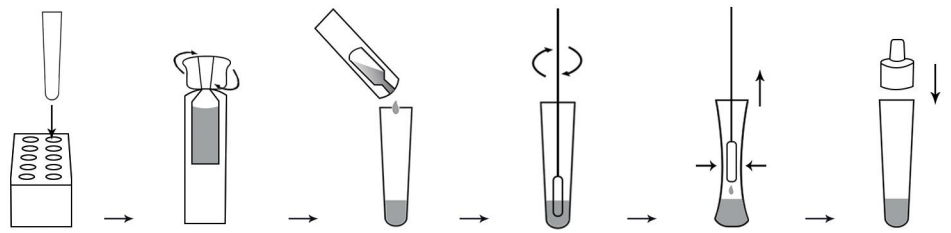

Roll the swab at least 5 times and leave it in the tube for 1 minute.

Remove the nasal swab while squeezing the sides of the tube to extract the liquid from the swab.

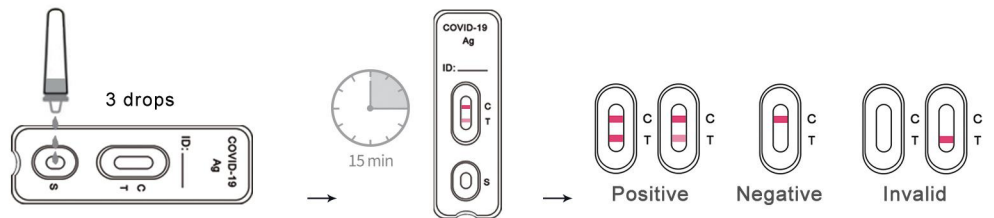

## ■ Features

